

EZEEKLEEN 2.5


Distillation Alternative

Model: 9000

COUNTERTOP INSTALLATION

Using Accessories Kit #9010
(Supplied with unit)

- Unit Dimensions
L x H x D
12 x 16 x 6
- Monitor Power Source
2 x AA Batteries

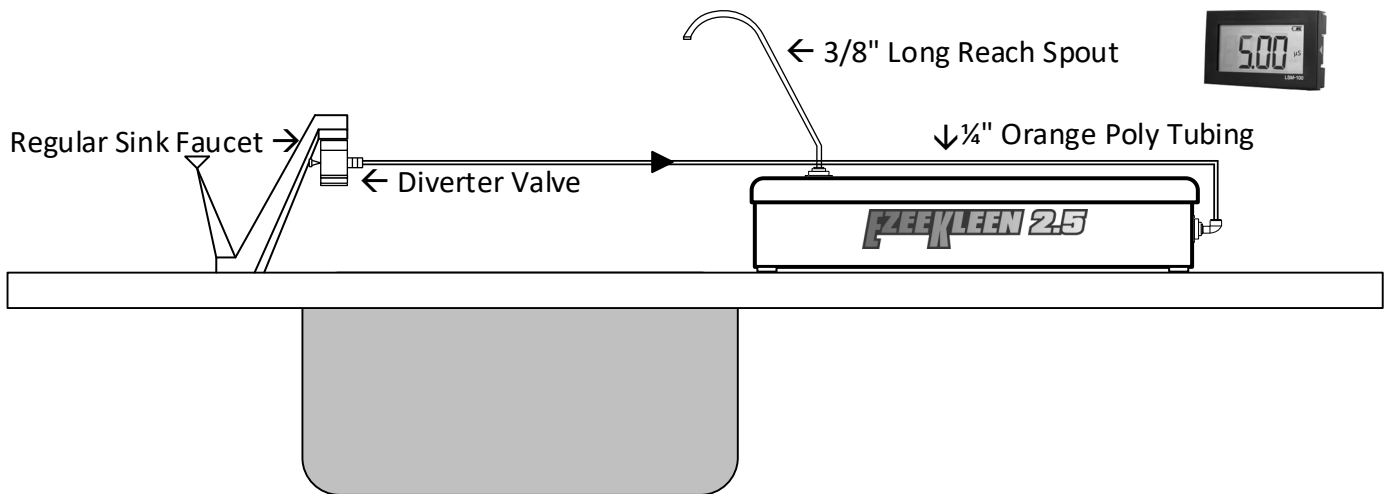

- Place Ezeekleen 2.5 on counter next to sink
- Remove screen from regular sink faucet
- Attach Diverter Valve to regular sink faucet (use one of the two adapters provided if necessary)
- Insert 1/4" quick connect elbow into top of unit
- Insert 3/8" Long Reach Spout into front of unit
- Mount Monitor in visible location and turn power on
- Ensure unit of measure is µS
- Run 8 oz (250 ml) of water through unit
- Ensure monitor reading is below 5.0 µS with water flowing
- Post "Protocol for Usage" (see page 16) at dispensing faucet and assign staff member responsible for system maintenance
- Ezeekleen 2.5 is now ready for use
- Change 9001 Ezeekleen 2.5 DI Cartridge (2 Pak) when monitor reads greater than 5.0 µS



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295