

# MAINTENANCE SUPPLIES

## TO ORDER YOUR SUPPLIES

1) Contact your authorized **EZEEKLEEN 2.5 HD** dealer.

For easy reference, enter your Dealer's name and contact information here:

---

---

---

Or

2) Contact **Oasis Dental Group** at:

(800) 338-6693 or (604) 538-1285

Fax: (604) 538-1295

Mail: 16147 - 8A Avenue, Surrey, BC V4A 8W8

Website: [www.oasisdentalgroup.ca](http://www.oasisdentalgroup.ca)

Email: [info@oasisdentalgroup.ca](mailto:info@oasisdentalgroup.ca)

### **Ezee Kleen 2.5 HD - Complete Unit - # 8000**

#### **De-ionization Cartridge - #8002-FR (includes 8006 flow restrictor)**

- change cartridge when monitor reads greater than 5.0  $\mu\text{S}$  with water flowing water is greater than 2.5 PPM (mg/L) of dissolved solids and is unsafe to use in autoclave. Change 8006 Flow Restrictor each time 8002-FR is changed.

#### **XL De-ionization Cartridge (Optional) - #8002-XL**

- change cartridge when changing 8002-FR

#### **Prefiltration Cartridge - # 8003**

- change annually
- not to exceed 2,000 gallons

#### **Reverse Osmosis Membrane - # 8004**

- change bi-annually (every two years)

#### **Ultraviolet Lamp - # 8005**

- ultraviolet cartridge will have a noticeable blue illumination when operating. Change the lamp if the illumination ceases, or
- change annually, not to exceed 9,000 hrs of continuous use

#### **Drainline Flow Restrictor—Part #8006**

- change when changing 8002-FR DI cartridge

## OPTIONAL FEATURES

- **XL De-ionization Install Kit - #8002-XL-KIT**

Order for initial XL DI Cartridge install (8002-XL DI Cartridge included)

- Optional tank sizes available
- Total Dissolved Solids Tester (Handheld)
- For water testing contact Oasis Dental Group or a local water testing facility.