

EZEKLEEN 2.5 HD

4 - STAGE ULTRA CLEAN WATER SYSTEM

Model: 8000

PROTOCOL FOR USAGE

When drawing water from the Ezee Kleen 2.5 HD ALWAYS ensure the following protocol is used:

- Tip: Photocopy and laminate or contact Oasis Dental Customer Support line for a free laminated copy of this page and post at the Ezee Kleen 2.5 HD dispensing faucet. Do not remove. Update chart each time cartridges are changed.
- Water is suitable for use in autoclaves, self contained dental unit water systems, ultrasonic baths, instrument cleaning and film developers
- Collect and store Ezee Kleen 2.5 HD autoclave safe water in clean containers marked “Autoclave Water Only” “No Tap Water” “Keep Lid On” “Store Only 48 Hours”
- Label the Dispensing Faucet "Autoclave Water - Always Check Monitor while Drawing Water"
- Disregard monitor when not dispensing water from the unit (ie. when unit sits idle)
- Check Ezee Kleen 2.5 HD Monitor each time water is drawn to ensure reading is less than 5.0 μ S.
- Regularly inspect the containers for cleanliness
- When storing filled containers ensure caps remain on
- Avoid storing or drawing water in an unclean environment
- Avoid storing the water more than 48 hours
- Ensure the working area around the dispensing faucet and especially the dispensing faucet itself remains clean
- Avoid running equipment that has the potential to create dust or particulate matter in the atmosphere when drawing water
- Use “Patient Scheduler” or sign up for our “Annual Maintenance Reminder Program” to schedule maintenance for Ezee Kleen 2.5 HD annually
- See page 11 of user’ manual for “Changing your Cartridges – Instructions”
- Failure to comply with manufacturer’s “Protocol for Usage” and below maintenance recommendations could result in water not suitable for intended use
- Completion of maintenance in accordance with below chart is only maintenance required
- Contact Oasis Dental Group Inc. customer support for any other maintenance

Cartridge	8002-FR DI Cartridge with drainline flow restrictor	Optional 8002-XL DI Cartridge	8003 Pre-Filter	8004 RO Membrane	8005 UV Lamp
Maintenance	Change when monitor reads greater than 5.0 μ S with water flowing	Change each time 8002-FR is changed	Change Annually	Change Bi-annually (Every Two Years)	Change Annually
Last maintained					
Changed by					

Cartridge Supplier:

Staff member responsible for maintenance of the Ezee Kleen 2.5 HD:

Questions? Contact customer support Mon – Fri, PST, 8:00am – 5:00pm.



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295

© 2009 - 2024 Oasis Dental Group Inc.™

www.oasisdentalgroup.ca