

# EZEKLEEN 2.5 HD

4 - STAGE ULTRA CLEAN WATER SYSTEM

Model: 8000

## 8002-XL DI CARTRIDGE INSTALL INSTRUCTIONS

*Tip: Have a towel available to absorb any water released by system.*

1. If possible have a full tank of water before installing the XL DI Cartridge
2. Turn off water supply under sink by closing Ezee Kleen 2.5 HD Shut Off Valve or cold water shut off valve
3. Close Ezee Kleen 2.5 HD Tank Valve
4. Open Ezee Kleen 2.5 HD Dispensing Faucet to relieve line pressure in the system
5. Remove Ezee Kleen 2.5 HD case cover
6. Widen slot located at the bottom right of the Ezee Kleen 2.5 HD to allow room for water lines (eg. drill 1" hole at the bottom of slot)
7. Mount XL DI Cartridge Bracket to surface (Side of Ezee Kleen 2.5 HD or convenient wall surface closest to unit) using (2) mounting screws provided or leave free standing beside Ezee Kleen 2.5 HD
8. Connect XL DI Cartridge Bracket to the XL DI Cartridge Receiving Cap using the (4) screws provided
9. Slide (2) Ezee Kleen 2.5 HD Slip in Elbow's into "IN" and "OUT" ports of XL DI Cartridge Receiving Cap
10. Remove blue protective cap from the top of XL DI Cartridge. XL DI Cartridge easily connects to Receiving Cap. Push XL DI Cartridge firmly in to Receiving Cap and rotate one quarter turn.
11. Remove ¼" black waterline that meets bottom of Ezee Kleen 2.5 HD DI Cartridge (Oasis ID: 8002-FR.) and ¼" elbow from ¼" union tee. *For instructions on connecting/disconnecting your cartridges see page 12 of the Ezee Kleen 2.5 HD User's Guide*
12. Insert ¼" elbow into the bottom of the Ezee Kleen 2.5 HD DI Cartridge (Oasis ID: 8002-FR)
13. Cut ¼" black water line provided with XL DI Cartridge to reach from the ¼" union tee (near the bottom of Ezee Kleen 2.5 HD DI Cartridge (Oasis ID: 8002-FR)) to the "IN" Port of the XL DI Cartridge Receiving Cap. Ensure a clean, square cut of water line then connect at both ends.
14. Cut remaining ¼" black waterline provided with the XL DI Cartridge to reach from the "Out" Port of the XL DI Cartridge Receiving Cap to the ¼" elbow at the bottom of the Ezee Kleen 2.5 HD DI Cartridge (Oasis ID: 8002-FR). Ensure a clean, square cut of waterline then connect at both ends
15. Water will now be directed through the XL DI Cartridge and then back to unit and thru Ezee Kleen 2.5 HD DI Cartridge (8002-FR)
16. Open Ezee Kleen 2.5 HD Tank Valve
17. Turn on water supply under sink by opening Ezee Kleen 2.5 HD Shut Off Valve or cold water shut off valve. Run until all air is purged from system
18. Run water from Ezee Kleen 2.5 HD Dispensing Faucet until you have a steady stream of water. Reading on water quality monitor should be < 5.00.
19. Close Ezee Kleen 2.5 HD Dispensing Faucet
20. Temporarily close Ezee Kleen 2.5 HD Tank Valve to fully pressurize system. Check for leaks and return Ezee Kleen 2.5 HD Tank Valve to open position
21. Unit is now ready for use. Change Ezee Kleen 2.5 HD XL DI Cartridge each time Ezee Kleen 2.5 HD DI Cartridge (Oasis ID: 8002-FR) is changed. (When monitor reads > 5.00 with water flowing)
22. The Ezee Kleen 2.5 HD 8002-XL DI Cartridge should reduce the 8002-FR Cartridge change frequency by 4 -5 times



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538-1295

© 2009-2024 Oasis Dental Group Inc.  
www.oasisdentalgroup.ca


Version: 03.04.24

# EZEKLEEN 2.5

## Distillation Alternative

Model: 9000

### 9001-XL DI CARTRIDGE INSTALL DIAGRAM

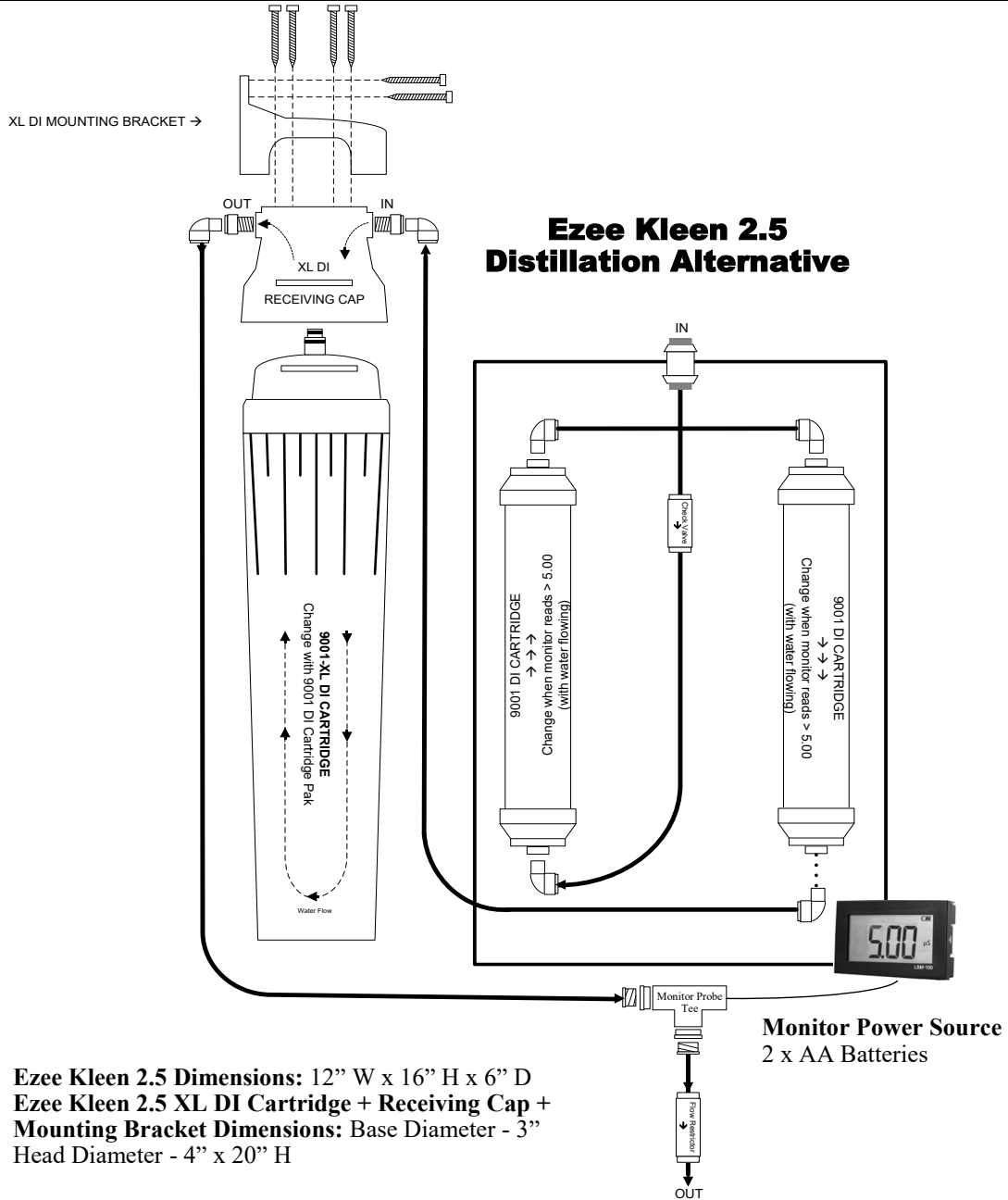

**Ezee Kleen 2.5 Dimensions:** 12" W x 16" H x 6" D  
**Ezee Kleen 2.5 XL DI Cartridge + Receiving Cap + Mounting Bracket Dimensions:** Base Diameter - 3" Head Diameter - 4" x 20" H



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538-1295

© 2009-2024 Oasis Dental Group Inc.  
 www.oasisdentalgroup.ca

Version: 03.04.24