

# **EZEEKLEEN 2.5**

## **Distillation Alternative**

### **Optional Sediment Cartridge**



## **Are you Maximizing the Life of your DI Cartridges?**

The Ezee Kleen 2.5's 9001 Deionization Cartridges are designed to remove dissolved solids such as iron, calcium, magnesium, sodium, etc.

Public utilities systems are vulnerable to larger particulate matter that can be characterized as sediment/suspended solids. These suspended solids could be sand, silt, flux, metal flaking from older plumbing (eg. Galvanized steel) organic matter, etc.

Sediment/Suspended Solids can reduce the flow rate of the Ezee Kleen 2.5 DI Cartridges and prematurely shorten their life.

Sediment/Suspended Solids enter the utilities water main system from such sources as:

- Water main system flushing programs
- Water main repair or expansion projects
- Water main breaks
- New tie ins for new commercial or residential construction
- Older buildings, especially those still having galvanized piping

Maximize the life of the Ezee Kleen 2.5's 9001 Deionization Cartridges by using our Ezee Kleen 2.5 9002 Sediment Cartridge

- Annual change recommendation

### **Fast Simple Installation**

Use the 9003 Sediment Cartridge Install Kit to install the Sediment Cartridge for the first time. Takes minutes to install.

For less than 10 cents a day!

**9002 Sediment Cartridge** (change annually)

**9003 Sediment Cartridge Install Kit** (includes one Sediment Cartridge)



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538-1295

# EZEEKLEEN 2.5

## Distillation Alternative


Model: 9000

### OPTIONAL SEDIMENT CARTRIDGE

#### Installing Optional Sediment Cartridge

*First time: Order Part #9003 Sediment Install Kit (Includes one cartridge.) Annually: Order Part #9002 Sediment Cartridge*

1. Attach brackets to the Ezee Kleen 2.5 Case using Sediment install kit.
  2. Move the 9001 DI Cartridge that is on the left to center brackets
  - Yearly Change out: Snap out and discard old Sediment Cartridge then follow steps 3 & 4**
  3. Remove protective white dust plugs from either end of the new Sediment Cartridge (see page 16 for quick connect fitting instructions) and snap in the new Sediment Cartridge into left brackets
  4. Attach tubing and components as shown in the below diagram
  5. Mark Sediment Cartridge for annual change out
- Tip: Use Patient scheduler to log reminder to change Ezee Kleen 2.5 Sediment Cartridge in one years time

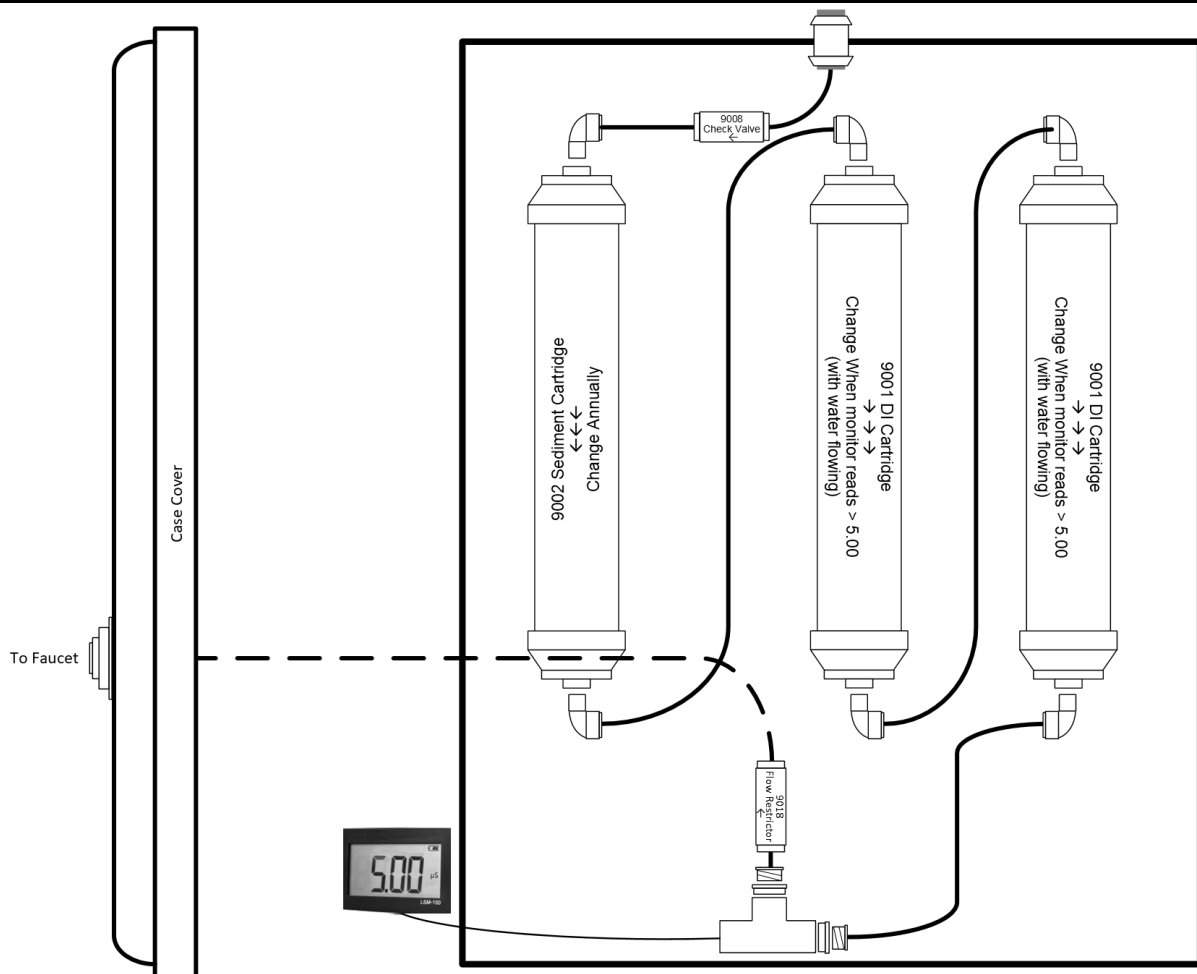

**Oasis**  
DENTAL GROUP INC.

15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295