

# EZEELKLEEN 2.5

## Distillation Alternative


Model: 9000

### WALL MOUNT INSTALLATION

Using Accessories Kit #9010  
(Supplied with unit)

1/4" Quick Connect Elbow →

- Place two screws in wall above sink and hang Ezee Kleen 2.5 on screws
- Remove screen from regular sink faucet
- Attach Diverter Valve to regular sink faucet (use adapters provided if necessary)
- Insert 1/4" quick connect elbow into top of unit
- Insert 3/8" Short Reach Spout into front of unit
- Mount Monitor in a visible location and turn power on
- Ensure unit of measure is  $\mu\text{S}$
- Run 8 oz (250 ml) of water through unit
- Ensure monitor reading is below 5.0  $\mu\text{S}$  with water flowing
- Post "Protocol for Usage" (see page 16) at dispensing faucet and assign staff member responsible for system maintenance
- Ezee Kleen 2.5 is now ready for use
- Change 9001 Ezee Kleen 2.5 DI Cartridge (2 Pak) when monitor reads greater 5.0  $\mu\text{S}$

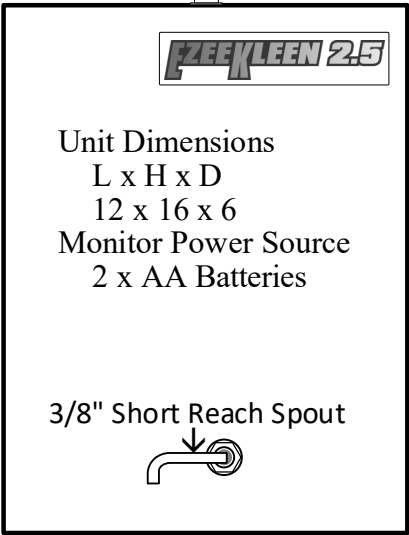

← 1/4" Orange Poly Tubing



Diverter Valve →

← Regular Sink Faucet



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295