

# EZEEKLEEN 2.5

## Distillation Alternative

Model: 9000

### 9001-XL DI CARTRIDGE INSTALL INSTRUCTIONS

*Tip: Have a towel available to absorb any water released by system.*

1. Turn off water supply (For under counter installation: Close Ezee Kleen 2.5 Shut Off Valve or cold water shut off valve and open Ezee Kleen 2.5 Dispensing Faucet to relieve line pressure in the system)
2. Remove Ezee Kleen 2.5 case cover
3. Widen slot located at the bottom left of the Ezee Kleen 2.5 to allow room for water lines (eg. drill 1" hole at the bottom of slot)
4. Mount XL DI Cartridge Bracket to surface (Side of Ezee Kleen 2.5 or convenient wall surface closest to unit) using (2) mounting screws provided or leave free standing beside Ezee Kleen 2.5
5. Connect XL DI Cartridge Bracket to the XL DI Cartridge Receiving Cap using the (4) screws provided
6. Slide (2) Ezee Kleen 2.5 Slip in Elbow's into "IN" and "OUT" ports of XL DI Cartridge Receiving Cap
7. Remove blue protective cap from the top of XL DI Cartridge. XL DI Cartridge easily connects to Receiving Cap. Push XL DI Cartridge firmly in to Receiving Cap and rotate one quarter turn.
8. Remove ¼" black waterline between bottom of right hand Ezee Kleen 2.5 DI Cartridge (Oasis ID: 9001) and right hand leg of Ezee Kleen 2.5 monitor probe tee. *For instructions on connecting/disconnecting your cartridges see page 19 of the Ezee Kleen 2.5 User's Guide*
9. Cut ¼" black water line provided with XL DI Cartridge to reach from the bottom of the right hand Ezee Kleen 2.5 DI Cartridge (Oasis ID:9001) to the "IN" Port of the XL DI Cartridge Receiving Cap. Ensure a clean, square cut of water line then connect at both ends.
10. Cut remaining ¼" black waterline provided with the XL DI Cartridge to reach from the "Out" Port of the XL DI Cartridge Receiving Cap to the open leg of the Ezee Kleen 2.5 monitor probe tee. Ensure a clean, square cut of waterline then connect at both ends
11. Water will now be directed through the XL DI Cartridge and then back to unit and thru Ezee Kleen 2.5 Monitor
12. Turn on water supply
13. Run water from Ezee Kleen 2.5 Dispensing Faucet until you have a steady stream of water. Reading on water quality monitor should be below 5.0 µS.
14. Check for leaks (For under counter installation: Close Ezee Kleen 2.5 Dispensing Faucet to pressurize system)
15. Unit is now ready for use. Change Ezee Kleen 2.5 XL DI Cartridge each time Ezee Kleen 2.5 DI Cartridge Pak (Oasis ID: 9001) is changed. (When monitor reads above 5.0 µS with water flowing)
16. The Ezee Kleen 2.5 9001 XL DI Cartridge should reduce the 9001 DI Cartridge Pak change frequency by 2.5 times.



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295

© 2009-2024 Oasis Dental Group Inc.™  
www.oasisdentalgroup.ca


Version: 03.05.21

# Ezee Kleen 2.5

## Distillation Alternative

Model: 9000

### 9001-XL DI CARTRIDGE INSTALL DIAGRAM

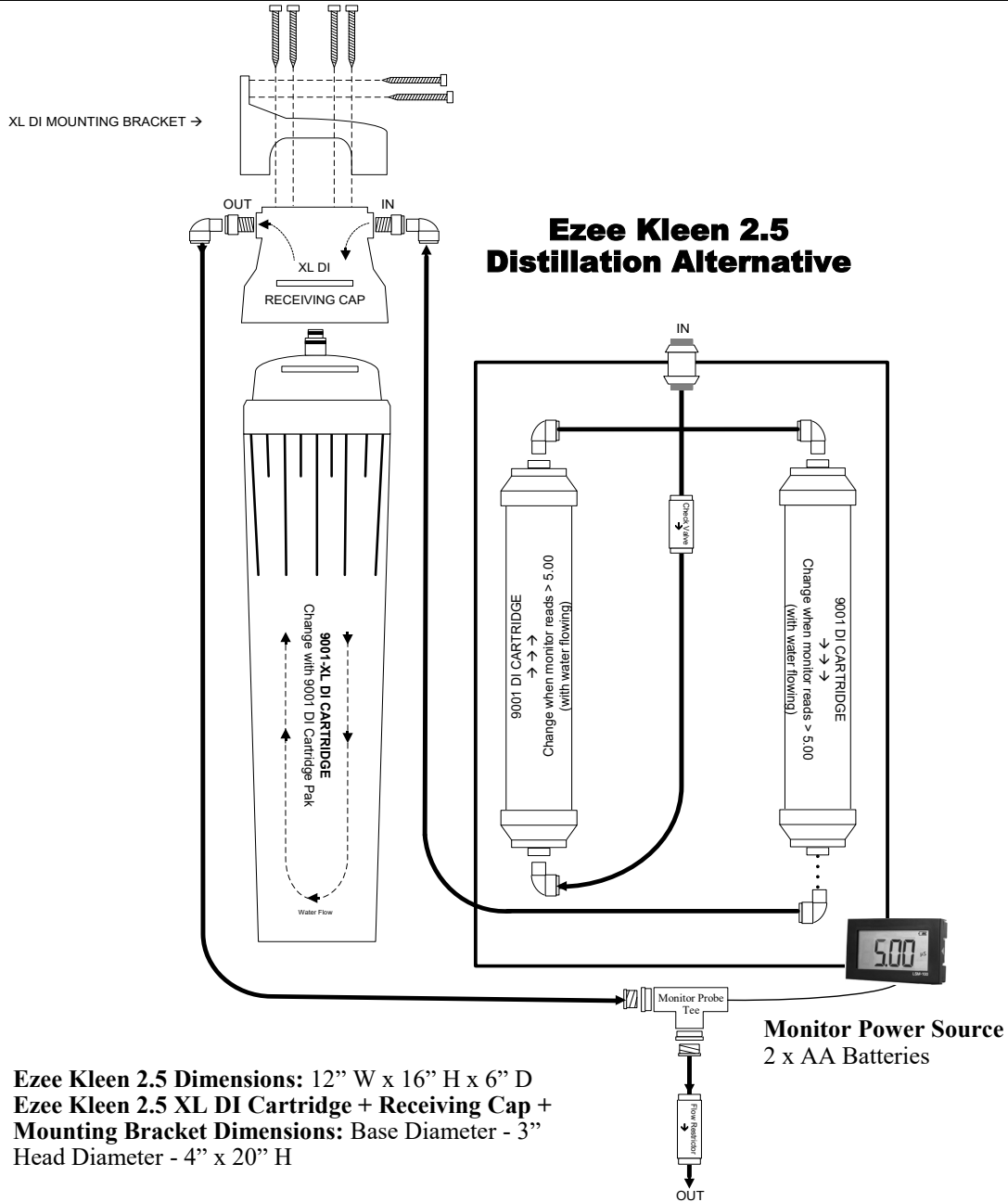

**Ezee Kleen 2.5 Dimensions:** 12" W x 16" H x 6" D  
**Ezee Kleen 2.5 XL DI Cartridge + Receiving Cap + Mounting Bracket Dimensions:** Base Diameter - 3" Head Diameter - 4" x 20" H



15531 - 24th Avenue, Unit 24, Surrey, BC V4A 2J3 • Ph: (800) 338-6693 • (604) 538-1285 • Fax: (604) 538 - 1295

© 2009-2024 Oasis Dental Group Inc.™  
 www.oasisdentalgroup.ca

Version: 03.05.24